

DRIED URINE COLLECTION

Do not leave dry samples out of the bag for more than 24 hours.

Samples **MUST** be completely dry before being placed back into the bag.


If you miss a collection time, collect the sample at the designated time the following day or example, if you miss the evening collection, collect in the evening the next day.

What if I lost the pink bag? Use a plastic sealable bag.

Reasons for sample rejection:

Samples that are still wet, not sealed in the bag, or too dilute (too much liquid consumed prior to collection) will not be processed, replaced or refunded. Samples over 30 days old will not be processed.


Your Kit Contains

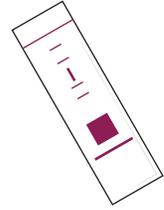

Collection Card



Paperwork

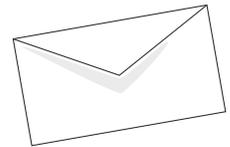

Return Mail Envelope x1

For your result please go to
<https://dashboard.labme.ai/>



Welcome To



Convenient Self-Collection Kit

Please read all instructions before handling collection device.

01

COLLECT SAMPLES

Step 1

At each collection time: Remove a collection card from the pink bag. Save this bag with the absorbent pouch. Write your name, the date and time of collection on the card prior to collecting the sample.



Step 2

To collect: Open the cover on the collection card. Fold it back and away from the filter paper. Avoid touching the filter paper. Saturate the filter paper by directly urinating onto it, or by collecting your urine in a clean cup and dipping the filter paper up to the top black dash line



Step 3

Hang or tape the collection card in a secure place to dry - like over a towel rack. Make sure that the urine-saturated filter paper does not touch anything.



02

FINISH & SEND

Step 4

Allow to dry completely for at least 6 hours.* Once dry, refold the collection card and place sample back into the baggie and seal. Make sure absorbent pouch is still inside. Follow this process for the remaining three samples.



Step 5

Complete all paperwork. Make sure all sections are filled in and your form is signed.



Step 6

Place the samples, the Test Requisition form and any additional paperwork back into the plastic test kit box. Make sure to include payment (if applicable). Snap test kit lid shut.



Step 7

Place the entire test kit box inside the return packaging. Affix the prepaid label or postage if applicable. Return using the appropriate carrier. Samples will remain stable under average shipping conditions, including over weekends and holidays.



Checklist

- Verify all information is correct on the Paperwork Form, sign and date where indicated. The Laboratory will **NOT** process your sample without the date of collection and your signature.
- Verify urine collection device is dry already

Seal envelope and drop in any US Mailbox. We will notify you when your results are ready. You can log into your dashboard at <https://dashboard.labme.ai>.

for assistance please contact support@labme.ai